

Canine Auditory Protection System (C.A.P.S.) Application Instructions

Placement of Ear Protection on Canine



Orient the C.A.P.S. as shown by the markings on top of the C.A.P.S.




the



Pull C.A.P.S. over the head of the canine. The front of the pads in the C.A.P.S. needs to be in front of the ear opening. The C.A.P.S. needs to be pulled back far enough to not block vision.

Continued on reverse of page

Continued on reverse of page

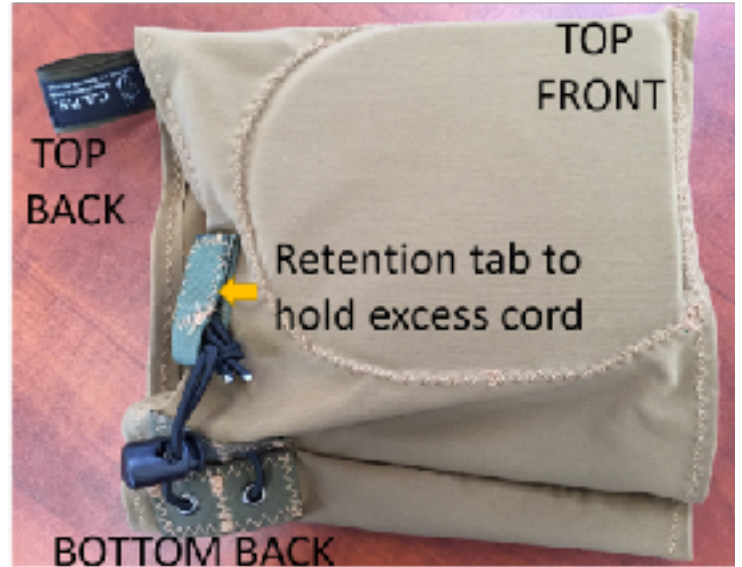

The underside of the C.A.P.S. has an opening in the rear to ease application over a muzzle. After positioning the C.A.P.S. snug up the cord and tuck it into the retention tab.

To remove C.A.P.S. pull on removal loop.



Continued on reverse of page

CAPS Sizes



Continued on reverse of page